

# THE GARGRAVE



**CE** Labelled standard chair product  
Codes and standards

1. Motor Actuator System meets EU – CE requirements for low voltage and EMC levels and correctly rated for the weight loading of the products.
2. All materials reach BS5852 part 1 and 2, test procedures for fire retardancy level Source 0,1 and 5 Woodcrib.
3. Products usage area to BS7176 low and medium hazard area of usage (0,1 – Domestic use – 0, 1 and 5 Woodcrib, NHS and Residential Care Requirements).
4. Warranty – motor actuator and chair shell 2 years. Subject to the terms and conditions of trading of Carewright Limited.

## IMPORTANT SAFETY NOTE

Children must NEVER be allowed to play with or left unsupervised near lifting seat or reclining back chairs at any time.  
Whenever possible remove the plug from the socket when product not in use. Please warn visitors with children of the potential danger of entrapment.

Carewright Ltd.	C3E
Vivary Buildings	
Spring Lane	
Colne	
BB8 9BD	
Telephone: 01282 869802	
Fax: 01282 865449	
Email: <a href="mailto:sales@carewright.co.uk">sales@carewright.co.uk</a>	
Web: <a href="http://www.carewright.co.uk">www.carewright.co.uk</a>	

Your Local Dealer:

Carewright

**TOTAL SEATING SOLUTIONS**



**THE GARGRAVE  
SINGLE MOTOR  
LIFT & RECLINE CHAIR**

**THE REVOLUTIONARY  
AIRLITE SEATING SYSTEM**

# THE GARGRAVE SINGLE MOTOR LIFT & RECLINE CHAIR

The **GARGRAVE** is a traditionally styled scroll-arm chair, combined with a modern electrically operated Rise and Recline mechanism.

The Low Voltage single Motor is operated by a user friendly Two Button Handset, making operation of the chair simplicity itself.

The **Lift** action is particularly beneficial for those who have problems in rising from the sitting position. Pressing the 'UP' button retracts the legrest and gently raises the chair to assist standing or sitting.

The **Recline** motion is operated by simply pressing the "DOWN" button. This gentle movement operates in two distinct stages: Firstly the leg rest slowly elevates, raising the users legs but still maintaining a comfortable upright seating position.

The second stage then slowly reclines the backrest for supreme comfort and relaxation. With three floor to seat heights and two seat depths, this chair offers a wide range of sizes to suit most needs



There is a choice of back styles for this chair: the Waterfall Back (pictured left) and the Button Back (pictured below left).



The waterfall back style features a three-section fibre filled backrest with each section individually adjustable to suit the user. This feature provides exceptional comfort and excellent pressure distribution properties.

The **GARGRAVE** can also be supplied with full length arms for extra support.

## AIRELITE SEATING SYSTEM

The **GARGRAVE** can now be supplied complete with the Revolutionary Airelite System, combining the very latest technology in **Pressure Relief** and **Dynamic Air Therapy** with a Quality Rise and Recline Chair.

### Specifications:

Side pocket fitted as standard, on left or right hand side.	Seat depth	18" 20"
Available in Button Back or Waterfall Back.	Seat width	18" 20" 23"
	Seat height	16" 18" 20"
	Back height	27" 30" 33"