


THE GRASSINGTON

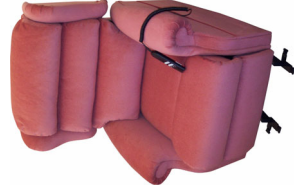

CE Labelled standard chair product
Codes and standards

1. Motor Actuator System meets EU – CE requirements for low voltage and EMC levels and correctly rated for the weight loading of the products.
2. All materials reach BS5852 part 1 and 2, test procedures for fire retardancy level Source 0,1 and 5 Woodcrib.
3. Products usage area to BS7176 low and medium hazard area of usage (0,1 – Domestic use – 0, 1 and 5 Woodcrib, NHS and Residential Care Requirements).
4. Warranty – motor actuator and chair shell 2 years. Subject to the terms and conditions of trading of Carewright Limited.

IMPORTANT SAFETY NOTE

Children must NEVER be allowed to play with or left unsupervised near lifting seat or reclining back chairs at any time.
Whenever possible remove the plug from the socket when product not in use. Please warn visitors with children of the potential danger of entrapment.

Carewright Ltd.	C2C
Vivary Buildings Spring Lane Colne BB8 9BD	
Telephone: 01282 869802	
Fax: 01282 865449	
Email: sales@carewright.co.uk	
Web: www.carewright.co.uk	

Your Local Dealer:

Carewright

TOTAL SEATING SOLUTIONS



**THE GRASSINGTON
TWO MOTOR
LIFT & RECLINE CHAIR**

**THE REVOLUTIONARY
AIRLITE SEATING SYSTEM**

THE GRASSINGTON TWO MOTOR LIFT & RECLINE CHAIR

The **GRASSINGTON** is a traditionally styled scroll-arm chair, combined with a modern electrically operated Twin Motor Rise and Recline mechanism.


The Low Voltage Motors are operated by a Four Button Handset, which can be safely stored in the built-in side pocket when not in use.

The extra motor allows independent control of leg rest and backrest giving the user an infinite number of seating and reclining positions to accommodate their individual needs.

A special feature of the **GRASSINGTON** is the incorporation of 'Tilt-in-Space' within the leg-rest action. Pressing the leg-rest 'up' button gently raises the

lower legs through 90°, with the back remaining upright. Further operation of the button raises the front of the seat and leg-rest, whilst the back moves in

harmony, creating a supremely comfortable and relaxing sitting position.

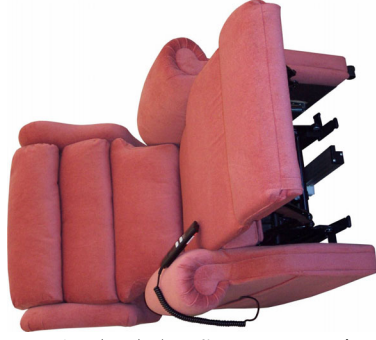

The waterfall back style (pictured) features a three-section fibre filled backrest with each section individually adjustable to suit the user. This feature provides exceptional comfort and excellent pressure distribution properties



AIRELITE SEATING SYSTEM

The **ILKLEY** can now be supplied complete with the Revolutionary Airelite System, combining the very latest technology in **Pressure Relief** and **Dynamic Air Therapy** with a Quality Rise and Recline Chair.

Specifications:

Side pocket fitted as standard, on left or right hand side.

Available in Button Back or Waterfall Back.

Seat depth 18",20"

Seat width 18",20",23"

Seat height 16",18",20"

Back height 27",30",33"

